

Ascend Full Face

replaceable AIR°gel cushion masks



Rise to a higher level of comfort and convenience with the Ascend full face mask: our first-ever full face mask featuring a replaceable cushion. Our patented AIR°gel replaceable cushion is easily detached and reattached for easy cleaning and cushion replacement, saving both time and money.

Ascend to a Higher Level of Comfort and Convenience

Flexible Spacebar™

Unique bendable design provides superior stability, holding a seal without contacting the skin.

Replaceable Cushion

Now you can replace the cushion without having to replace the whole mask. Available in three sizes.



AIR°gel

Maximum comfort, minimum weight. Our patented AIR°gel minimizes pressure points and skin irritation.

Advanced Cushion Technology™ (A.C.T.)

Our ergonomic cushion technology delivers an effective seal for the most comfortable fit.

Quick-Release Headgear


Lightweight, breathable headgear slides off easily with one hand.

360° Rotating Elbow

For optimal tube positioning and easier mobility

Active Headgear Connector™

Moves as the patient moves for a reliable seal.

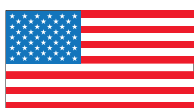

Made in the USA





ABABIL
PEOPLE | PASSION | PRODUCTS

What makes our Ascend Full Face Mask different?

- ✓ Our easy-to-use replaceable cushion allows you to extend the life of your mask.
- ✓ Flexible Spacebar is designed to provide superior stability without contacting the skin.
- ✓ Active Headgear Connector moves when the patient moves to help maintain an effective seal.
- ✓ Available in three sizes with simple and breathable quick-release EZFit headgear.

Advanced Cushion Technology(A.C.T.)

Our patented AIR^ogel with an ergonomic design and luxurious feel reduces pressure while maintaining an effective seal.



Easy-to-Replace Cushion



Disassembly: Just squeeze the sides of the cushion in to release and snap in to replace.



Assembly: Place the top alignment pin in the notch and push the bottom ones in until the pins fit securely.

Ascend Full Face Masks with EZFit Disposable Headgear Specifications:

Product Number	Description	Leak Port	Entrainment Valve	Cushion Size	Mask Weight	Mask Dimensions
50825	Mask System & headgear	✓	✓	S, M, & L	NA	NA
50834	Mask System with attached headgear	✓	✓	S, M, & L	NA	NA
50813	Mask & headgear	✓	✓	Small	94.5g	3.92"D x 3.86"W x 7.00"H
50814	Mask & headgear	✓	✓	Medium	103.4g	4.16"D x 4.20"W x 7.00"H
50815	Mask & headgear	✓	✓	Large	107.8g	4.24"D x 4.30"W x 7.00"H
50801	Mask with attached headgear	✓	✓	Small	94.5g	3.92"D x 3.86"W x 7.00"H
50802	Mask with attached headgear	✓	✓	Medium	103.4g	4.16"D x 4.20"W x 7.00"H
50803	Mask with attached headgear	✓	✓	Large	107.8g	4.24"D x 4.30"W x 7.00"H
50207	AIR ^o gel Cushion	✓	✓	Small	94.5g	3.92"D x 3.86"W x 7.00"H
50208	AIR ^o gel Cushion	✓	✓	Medium	103.4g	4.16"D x 4.20"W x 7.00"H
50209	AIR ^o gel Cushion	✓	✓	Large	107.8g	4.24"D x 4.30"W x 7.00"H
55051	EZFit Headgear	NA	NA	NA	NA	NA